

Proper Omnilux® lens care instructions

"80% of lens scratches"
occur by cleaning eyewear incorrectly



Always use warm water and mild soap without additives to clean your lenses. Never clean them dry!

You may also use an approved lens cleaning solution provided by your eye-care professional



After washing, dry your lenses with soft lint-free cloth. You may also use a microfiber cloth specially designed for your eyewear.

Never use glass cleaner or other harsh home cleaners including alcohol. Avoid all chemicals like hair spray, perfume or cologne which could cause damage.



- NEVER use facial tissues or paper towel to dry your lenses as they may scratch.
- NEVER leave your eyeglasses in direct sunlight or in a hot car. The heat can damage your lenses.
- NEVER leave your eyeglasses facing downward.

Always keep your eyeglasses in the case when you are not using them!

OmniLux[®] Ai

Fitting Height is not needed
But Tracing data is required
for the Ai in order to calculate
the individual design placement

**There are two types of
Omnilux[®] NAL[®] lenses**
**AND THE ONLY DIFFERENCE IS IN
HOW THE DESIGN IS PLACED ON THE LENS**

OmniLux[®]

Fitting Height is required
And no Tracing data is needed
as the Ai is not involved in
the individual design placement



QLDS Inc.
9191 132nd Ave N,
Largo, FL 33773

qldsamerica.com
info@qldsamerica.com
linkedin.com/company/qlds

OmniLux[®]



NAL[®] Natural Accommodation Lens

What is Omnilux[®]?

When you receive your brand new Omnilux[®] lenses you will begin a new journey never experienced before with any other multifocal lens. Omnilux[®] requires no adaptation period and gives wearers the feeling they are wearing Single Vision lenses with the performance of a multifocal


*Enjoy natural
youthful
vision again™*

NAL[®]


NOT A PROGRESSIVE LENS, BUT 1ST TRUE MULTIFOCAL

Lifestyle Activities


NAL® FOCAL LENGTH BACKBONE DECELERATION RAMP

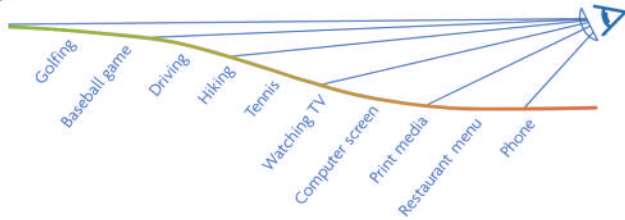

NAL®
Natural Accommodation Lens

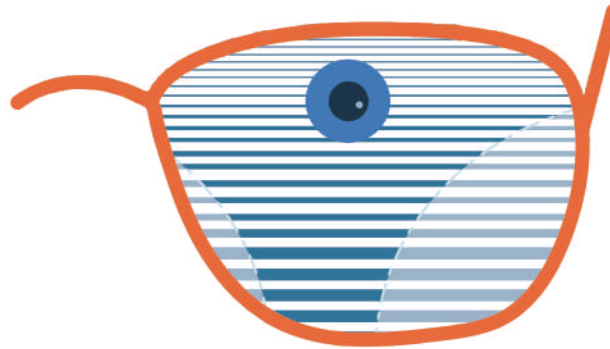

PAL
Progressive Addition Lens

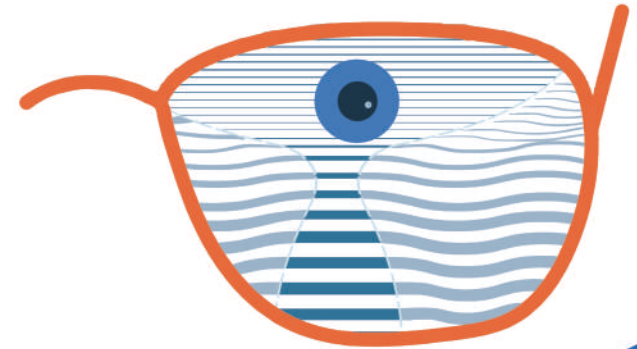

Professional ✓ Leisure ✓ Daily ✓

Natural comfortable vision for all activities

Omnilux®
NATURAL ACCOMMODATION LENS - NAL®

Thanks to Omnilux® lenses you can perform all your daily lifestyle activities with natural clear vision.

No adaptation period allows you to enjoy your new glasses immediately.

Omnilux® lenses are also suitable for your sunglasses.

Ask your optician for more information

Omnilux® B E N E F I T S

- No adaptation period
- No peripheral swim effect
- Provides clear, natural vision in all distances
- No fitting height required

Omnilux® lenses are designed **to be worn all day**, and are recommended to all multifocal wearers.

Progressive R E A L I T Y C H E C K

- Adaptation period necessity
- Peripheral swim effect
- Supports clear, limited vision in some distances
- Fitting height is required

Driving glasses Glasses ...
Glasses for work Reading glasses
Glasses ... Glasses for Pilots
Glasses to play Golf Glasses ...